


# NewCare Bariatric Bed

## QUICK REFERENCE SHEET

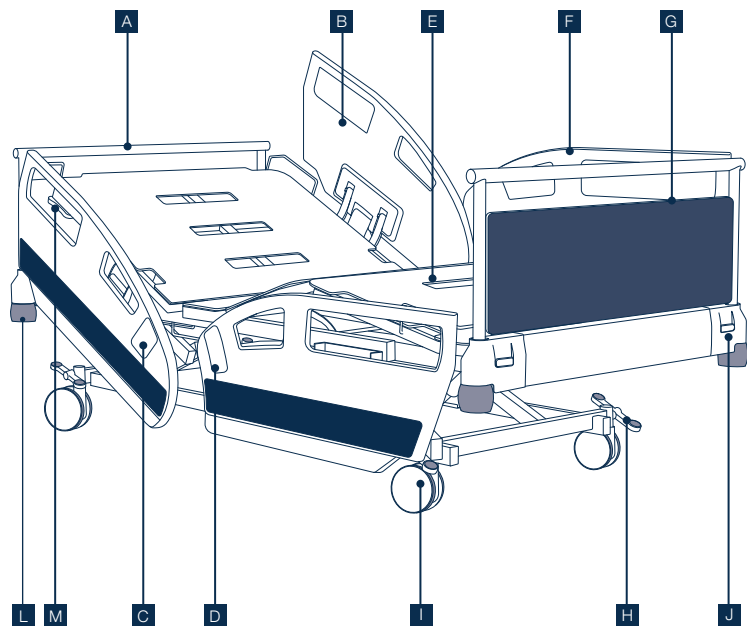

### BED'S COMPONENTS

A	Removable stainless steel headboard	G	Removable stainless steel footboard
B	Backrest side rails	H	Braking pedal
C	Backrest tilting indicator	I	Double wheels (Ø150 mm.)
D	Trend / Reverse Trend tilting indicator	J	Headboard and Footboard unlocking
E	HPL mattress platform	L	Protection discs
F	Legrest side rails	M	Emergency CPR

### OTHER FUNCTIONS

#### Side Rails (1):


- Raising the rail: Pull the rail upwards until it is immobilised in the highest position.
- Lowering the rail: Pull the handle found underneath the rail outwards. The rail will automatically lower.

#### Emergency CPR (2):


- Activate either of the two levers located under the headrest module, accompanying it in turn as it lowers.

#### Centralised braking system (3):



- Braking the bed: Pedal in low position.
- Moving the bed: Pedal in intermediate position.
- Activating steering wheel: Pedal in raised position.

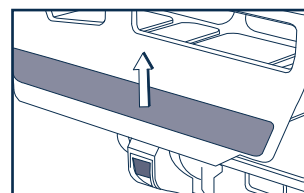


Raising the rails

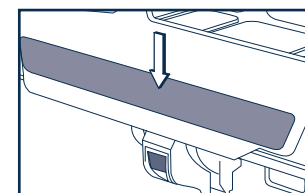


Lowering the rails

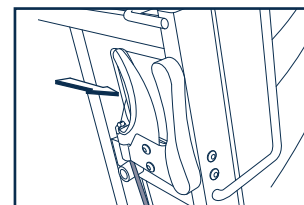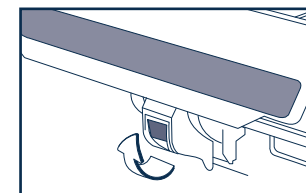


Emergency CPR

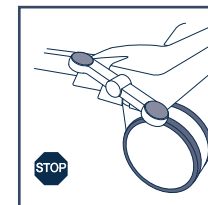

Brake

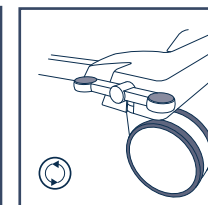

Free movement

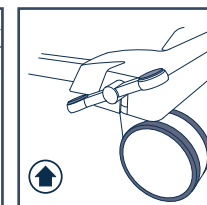

Steering wheel

### CLEANING

Daily cleaning and disinfection with soapy water:

- Rails. Head board. Foot board.

Cleaning and disinfection after a patient leaves or before a transfer:

- Mattress platform area. Chassis of the bed.

Thorough cleaning and disinfection every two months:

- Support and lifting area of the bed.

### MAINTENANCE

Always check the correct state of:


- Electrical wiring
- Wheels.
- Braking mechanisms.




## CONTROLS

### INSTRUCTIONS OF USE


A	Rise backrest	E	Rise legrest	I	Reverse Trend	P	Emergency CPR
B	Lower backrest	F	Lower legrest	J	Trend	U	Lock / Unlock backrest section
C	Rise mattress platform	G	Rise all sections simultaneity ( <i>autocontour</i> )	L	Indicator Led of locked movements	V	Lock / Unlock backrest section
D	Lower mattress platform	H	Lower all sections simultaneity ( <i>autocontour</i> )	N	Cardiac chair position	W	Lock / Unlock mattress platform

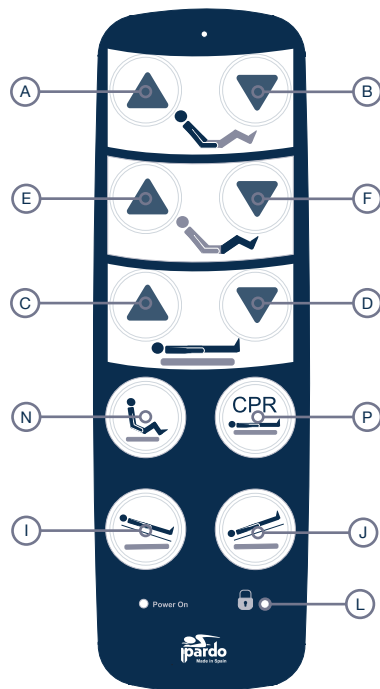

Patient control with locking

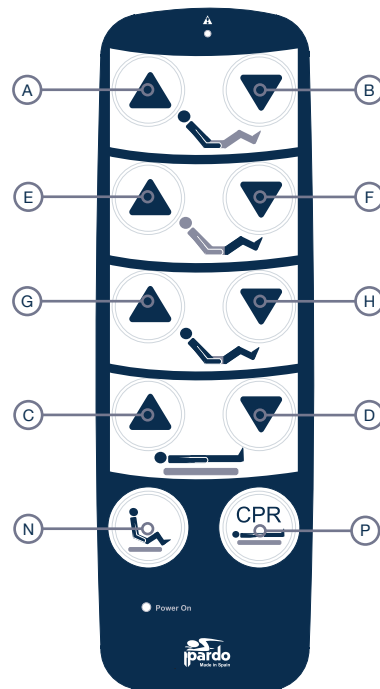

Patient control without locking

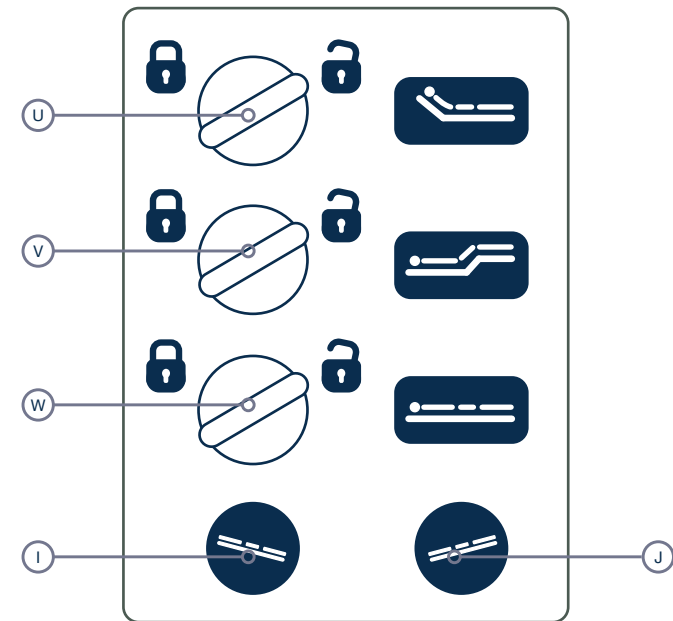

Locks nursing control



#### SAFETY WARNINGS

- The maximum duration of a continuous manoeuvre should not exceed 2 minutes, as damage may be caused to the equipment.
- Before raising and lowering the moving parts of the mattress platform, with the other parts of the bed or accessories, it is essential to check that there are no obstacles between them.
- The Trendelenburg and reverse Trendelenburg movement should always be performed under medical supervision.
- When the bed is in extended position, do not sit or lean on the foot part of the bed, since it may cause a bed dump.
- Do not use CPR bilateral actuating levers to raise the backrest of the bed, it may cause raise motor breakdown and makes a bed dysfunction.

