

BED'S COMPONENTS



A	Screwed headboard	F	Wooden double slat beds
B	Steel plate wheels Ø12,5 cm.	G	4 tube handrails
C	Wired control	H	Accessories support
D	Braking pedal	I	Rounded bed base in footboard area
E	Screwed footboard		

SIDE RAILS



Four tube side rails:

- Rise side rails: Pull of the side rail towards headboard up to hearing "click" from the lower handle, then the side rail is locked in highest position.

- Lower side rails: Unlock side rail lower handle pulling of it down. Fold the said rail towards footboard taking it until lowest position where it remains locked.

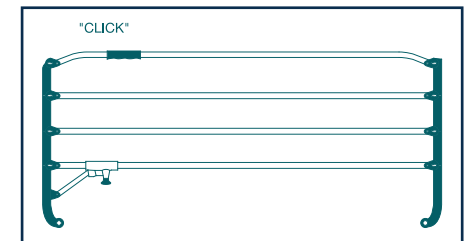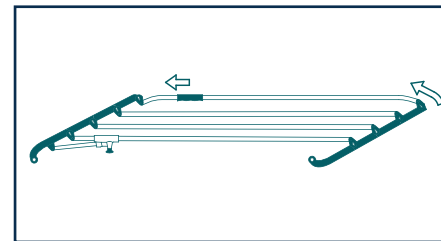

Rising the rails

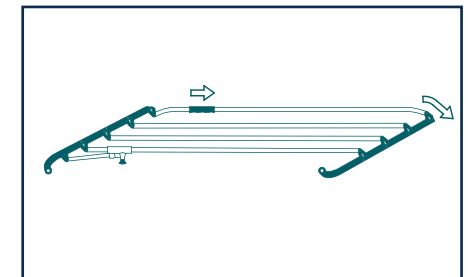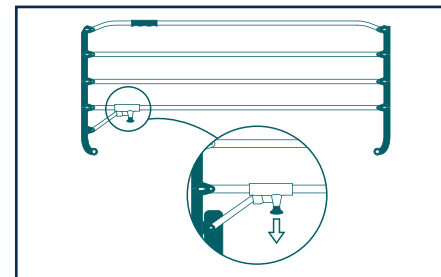

Lowering the rails

CLEANING

Daily cleaning and disinfection with at least 1000ppm concentration of chlorine solutions:

- Rails, headboard, footboard.

Cleaning and disinfection after a patient leaving or before a transfer:

- Mattress platform area, chassis of the bed.

Deep cleaning and disinfection every two months:

- Support and lifting area of the bed.

MAINTENANCE


Always check the correct state of:

- Electrical wiring
- Wheels.
- Braking mechanisms.


OTHER FUNCTIONS

Braking pedal (1):


- Braking the bed: Pedal in low position.
- Moving the bed: Pedal in the middle position.
- Steering wheel actining: Pedal in raised position.

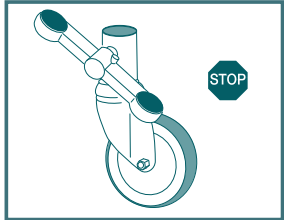

Brake

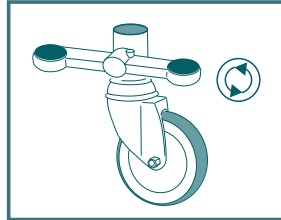


Free movement

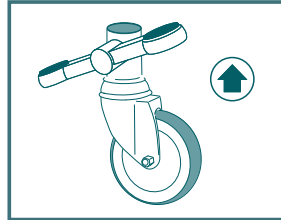


Steering or 5th wheel

Range of mattress platform (2):

- Slat beds of highly resistant vaporized beech wood grouped two by two by flexible thermo-rubber suspensions.
- Ergonomic ABS bed base panels divided into four sections, made of sanitary plastic.

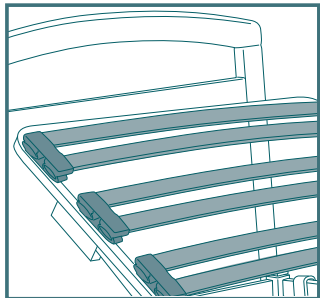


Wooden double slat beds

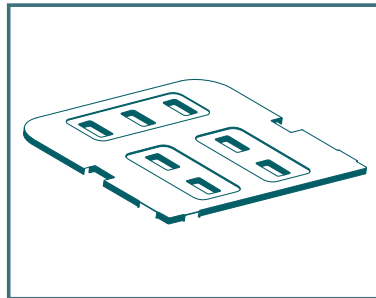

ABS bed base panels

CONTROLS

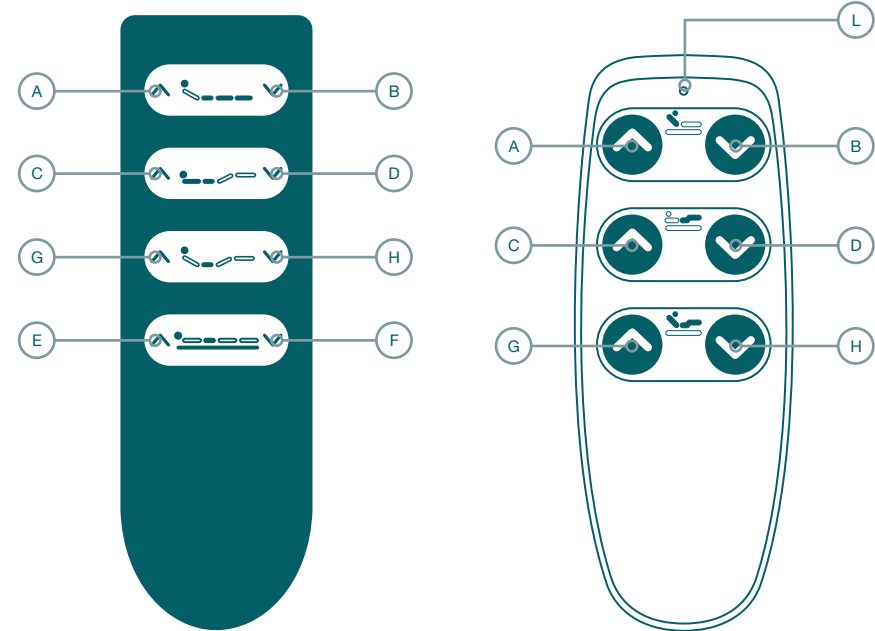

Wired patient control for bed with trolley

Wired patient control for bed with legs

INSTRUCTIONS OF USE

A	Rise backrest	F	Lower mattress platform
B	Lower backrest	G	Rise backrest and legrest simultaneously (<i>autocontour</i>)
C	Rise legrest	H	Lower backrest and legrest simultaneously (<i>autocontour</i>)
D	Lower legrest	L	Movement indicator Led
E	Rise mattress platform		



SAFETY WARNINGS

- Before raising and lowering the moving parts of the mattress platform, with the other parts of the bed or accessories, it is essential to check that there are no obstacles between them.
- The maximum duration of a continuous manoeuvre should not exceed 2 minutes, as damage may be caused to the equipment.

